

> SILICONE DRAINS



Code	Description	Box Qty.
------	-------------	----------

SILICONE DRAINS DOUBLE PACKED

- Soft, flexible and inert for patient comfort and faster healing.
- Closed Wound Drain Systems are used to remove blood and excess fluid from wounds, so as to prevent or minimise haematomas and eliminate dead space.
- Promotes the approximation of skin flaps and reduces the potential of post-op infections.
- Closed Wound Drain Systems are composed of 2 components - Surgical Drain and Evacuators/Reservoirs.
- Trocars can be included or can be attached during the surgical procedure.

SILICONE BULB

- Silicone drainage Bulb is compatible with the following drains below: Flat Drains, Fluted Drains, Round Drains.
- Squeeze the Bulb and then attach to the drain for suction.
- As the Silicone Bulb expands, it creates a vacuum to a maximum of 125mmHg pressure, and slowly sucks fluid and blood from the wound site, preventing Haematomas forming.
- The Bulb is usually removed 1 - 2 days after the operation.

Code	Description	Box Qty.
DEF1400	SILICONE BULB 100ml/cc	25



Code	Description	Box Qty.
DEF1400-400	SILICONE BULB 400ml/cc	25

FLAT SILICONE DRAINS

- Hubless drains
- Soft and comfortable silicone flat drains with multiple drainage holes 36 x 2 Holes, to enhance the fluid removal in a wound in the patient.
- Ergonomic Flat shape enabling easier drain placement.
- Indicated for Abdominal Surgery, and suitable for use as a Pleural Space Drain.

Code	Description	Box Qty.
11100200780DS	WHITE X/R 7mm x 190mm	10



Code	Description	Box Qty.
------	-------------	----------

11100201080DS	WHITE X/R 10mm x 190mm	10
---------------	------------------------	----

**FLUTED SILICONE DRAINS**

- Flexible silicone drains with four channel unique H shape.
- This drain shape allows for greater multi-channel drainage, and also ensures ease of removal due to no solid silicone outer drain perimeter.
- Inner core of H design offers drain strength and flexibility.
- Suitable for Plastic Surgery particularly Breast, Abdominoplasty and Rhytidectomies. Also used extensively in Cardiac Surgery. Useful as an adjunct to the following surgical procedures: Orthopedic, Neurological, Gynecological, Head and Neck.

**round**

11003131080DS	CLEAR FULL FLUTED 10FR with Trocar	10
11003021080DS	CLEAR FULL FLUTED 10FR	10



Code	Description	Box Qty.
11003831080DS	WHITE WITH HUB 10FR	10
11003131580DS	CLEAR FULL FLUTED 15FR with Trocar	10
11003131980DS	CLEAR FULL FLUTED 19FR with Trocar	10



11003811980DS	WHITE 19FR with Hub	10
---------------	---------------------	----



11003122480DS	CLEAR FULL FLUTED 24FR Hubless flat white	10
11101310780DS	3/4 FLUTED 7mm with Trocar, with Hub	10
11101311080DS	3/4 FLUTED 10mm with Trocar, with Hub	10
11101410780DS	FULL FLUTED 7mm with Trocar, with Hub	10
11101411080DS	FULL FLUTED 10mm with Trocar, with Hub	10

Code	Description	Box Qty.
------	-------------	----------

SILICONE ROUND CLEAR DRAINS X-RAY

- Drain consists of 5 side holes with a drain length of 50cm.
- Used for General Surgery, Chest and Neck Surgery.



11000301080DS	10 FR 1.6mm x 3.2mm	10
11000301280DS	12 FR 2mm x 4mm	10
11000301580DS	15 FR 3mm x 5mm	10
11000301880DS	18FR 4mm x 6mm	10
11000302180DS	21FR 4.5mm x 7mm	10
11000302480DS	24 FR 5mm x 8mm	10
11000302780DS	27FR 6mm x 9mm	10
11000303080DS	30 FR 7mm x 10mm	10
11000303680DS	36 FR 8mm x 12mm	10

CAPILLARY ACTION DRAINS - PASSIVE WOUND DRAINAGE MULTILUMEN DRAINS X-RAY

- Clear Drains with varying lumens according to drainage required.
- Lumens can be separated out and positioned as necessary. The drainage takes place both through, under and over the lumens.

DEF1110	2 LUMENS x 12.5cm	10
DEF1076	5 LUMENS x 25cm	10



DEF1788	9 LUMENS 2.5cm x 20cm	10
DEF1024	9 LUMENS 2.5cm x 25cm	10
DEF1750	16 LUMEN x 20cm	10
DEF1751	16 LUMEN x 40cm	10

EASY FLOW DRAINS -

- 300mm Drain Length Radio-opaque Striped Clear drain.
- Drain incorporates inner "teeth" to ensure open drainage.

21740632	FLAT 40mm	10
21706632	ROUND 6mm	10